zetaflow

polysiloxane (condensation silicone)
impression material

zetaflow putty

zetaflow light

zetaflow catalyst

UT13385-30/04 Last Update 04/2015

Zhermack S.p.A.

Via Bovazecchino, 100 | 45201 Badia Polesine (RO) Italy

T +39 0425 597611 | F +39 0425 53596

info@zhermack.com | www.zhermack.com

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SOLO PER USO DENTALE

HYDROCOMPATIBLE CONDENSATION SILICONES (POLYSILOXANES) FOR HIGH-PRECISION IMPRESSIONS

USE

Zetaflow Putty: Polysiloxane for the first impression characterised by initial flow in the working phase and high rigidity after setting. Recommended for the two stage technique (double impression).

Zetaflow Light: Low viscosity, extremely fluid, hydrocompatible, polysiloxane. Recommended for the two stage technique (double impression).

Zetaflow Catalyst: Specific catalyst for Zetaflow Putty and Zetaflow Light.

IMPRESSION TRAY: All rigid types; Hi-Tray (Zhermack) stainless steel impression trays are particularly recommended. Use Universal Tray Adhesive (Zhermack) with impression trays without perforations or rim locks.

ZETAFLW PUTTY

INSTRUCTIONS FOR USE:

Dosage: Use the measuring scoop to dose the required quantity of ZETAFLW Putty and spread it out over your hand (note: for the correct dosage the putty should be level with the top of the measuring scoop). Press the rim of the measuring scoop onto the putty once for each measure dosed. For each measure of material, apply one strip of ZETAFLW Catalyst as long the measuring scoop (4 cm.) (1).

Mixing: Fold and knead the putty energetically with your fingertips for about 30 sec., until the colour is even, without any stripes (2).

Spread out the putty over the impression tray. The working time including mixing time is about 1 min. 15 sec. (3). Put into the mouth and wait until it is set, which will take about 3 min. 15 sec. (4).

ZETAFLW LIGHT

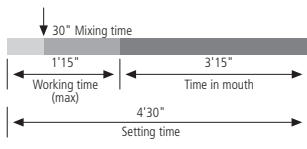
INSTRUCTIONS FOR USE:

Dosage: Extrude the required quantity, observing the dosage scale on the mixing pad. Then dose a quantity of ZETAFLW Catalyst equal to the length of the fluid (5).

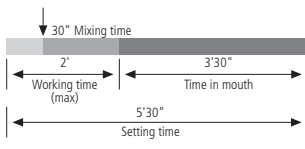
Mixing: Mix energetically with the spatula, and then carefully spread the mixture out, pressing it over the mixing block to eliminate any air which may have been incorporated into the mixture. Repeat this operation until the mix is even in colour. Mixing time should ideally be about 30 sec. (6). Apply the mixture to the impression tray with a spatula or a syringe for elastomers. Working time including mixing time is about 2 min. (7).

Put into the oral cavity and wait for the mix to set, which will take about 3 min. 30 sec. (8).

TIMES FOR CLINICAL USE

ZETAFLW PUTTY

ZETAFLW LIGHT

WARNING: In sensitive subjects, polysiloxanes may cause irritation or other allergic reactions.

CLEANING AND DISINFECTION OF THE IMPRESSION

After rinsing well under running water, the impression can be immediately disinfected. Ideal disinfecting is achieved by immersing the impression in **Zeta 7 solution** or spraying it with **Zeta 7 spray** (Zhermack). When using other disinfectants, always follow the manufacturer's instructions.

CASTING MODELS: the impression must be cast 30 min. to 72 hours after setting. To reduce surface tension and therefore the risk of bubble formation (with dental stone materials) we recommend treating the impression by spraying with Zhermack Tensilab. The material is compatible with all quality commercially available dental stone. We recommend Zhermack gypsum, type 4 die stone: **Elite Rock/Elite Rock Fast** (Zhermack) and type 3: **Elite Model/Elite Model Fast** (Zhermack).

WARNING / CAUTION

- Before making the wash impression, remove all remaining traces of oral fluids by rinsing thoroughly and drying.
- The times given are for mix starting at 23°C/73°F. High temperatures or overuse of catalysts will speed up setting, whereas low temperatures or insufficient use of catalysts will slow it down.
- Close all setting substances properly after use.
- Use of gloves is recommended. Prevent catalyst from coming into contact with skin and eyes; wash well with soap and water in the event of accidental skin contact. In the event of eye contact, rinse thoroughly with running water and seek medical attention. If substance is ingested, seek medical attention immediately.
- Do not stain garments.

GALVANISATION: The impressions may be galvanised with copper or silver.

CONSERVATION: Guaranteed for 36 months at 5°C to 27°C (41°F – 80°F)

TECHNICAL DATA	ZETAFLW PUTTY	ZETAFLW LIGHT
ISO 4823 / ADA 19	Type 0 Putty Consistency	Type 3 Light-bodied Consistency
Mixing time	30"	30"
Total working time*	1'15"	2'
Time in mouth*	3'15"	3'30"
Setting time*	4'30"	5'30"
Strain in compression (min - max)	2-5%	2-8%
Recovery from deformation	>98%	>99%
Linear dimensional change 24 hours	<-0,2%	<-0,7%

*The times mentioned must be intended from the start of mixing at 23°C – 73°F. Higher temperatures reduce the times, lower temperatures increase them.

IMPORTANT REMARKS: Consulting services on the use of our products, provided in any form, be that verbal, written or through demonstrations, are based upon the current state of dentistry and of our know-how. They are to be considered as non-binding information, also in relation to possible third party claims, and do not exonerate the user from personally checking whether the product is suitable for the planned use. Use and application by the user take place under no possible control by the Company, and are therefore under the user's responsibility. Possible liability for damages is limited to the value of goods supplied by the Company and utilized by the user.

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Zetaflow Catalyst: Specific Catalyst for Zetaflow Putty and Zetaflow Light.

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