CEREC Optispray

Deutsch

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Produktbeschreibung

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Instructions

Product description

CEREC Optispray enables easy and perfect optical impressions of CAD/CAM restorations. CEREC Optispray is designed for (chairside) intraoral application. Due to its optimized dispensing and a special nozzle, it always ensures homogeneous coating of the object.

Composition

Pigment suspension in ethanol; fluorinated hydrocarbon

Flavor: peppermint

Application

Intraoral application (prepared and unprepared teeth)

Application restrictions

All surfaces must be cleaned and dried prior to application. Do not use on

heavily bleeding wounds covering a large area.

General note

Shake before using!

Failure to observe the specified application restrictions and procedure may result in measuring errors.

Side effects

No side effects are known to date. Allergic reactions may occur in individual cases. CEREC Optispray should not be used if it is suspected or known that the patient may be allergic to any of its ingredients.

Application procedure

- The surface to be coated must be thoroughly cleaned and dried.
- Attach spray head with cannula/nozzle and push the latter into the spray head until the colored mark on the cannula/nozzle disappears in the spray head. Prior to each use, check whether the fitted cannula/nozzle is firmly mounted by slightly pulling it. Shake well before use.
- The cannula/nozzle can be rotated as required to enable optimal coating from all directions. When coating objects in the posterior tooth region, you can use the stabilizing tube to push the patient's lip to one side without deforming the cannula (the stabilizing tube is included only in the 200 ml package).
- The spray must be carefully aimed to ensure that the area for the impression properly coated.
- The spray nozzle should be held approx. 10-15 mm away from the object.
- Take an optical impression in the usual manner.
- As soon as the 3D impression has been taken, the surface should be cleaned with air/water spray.
- The cannula/nozzle should be replaced after each use.

CAUTION:

The can is pressurized.

Protect against direct sunlight and temperatures exceeding 50°C.

Do not open by force or burn even after use.

Keep out of the reach of children.

CAUTION.

Federal Law (USA) restricts sale of this device to or on the order of a physician, dentist, or licensed practitioner.

Preparation for use

The material was developed for use in the dental area and must be applied according to the operating instructions. The manufacturer assumes no liability for damage ensuing from misapplication or improper use of this product. Furthermore, the user is obliged to test the material for suitability and applicability for the intended purposes on his or her own responsibility before using it, especially if these purposes are not included in these operating instructions.

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