# STODENT

dental stone

## EN - Dental stone

**Product characteristics:** dental stone

### Method of use

**Hand mixing:** Using the proportions listed in the table below progressively pour gypsum into water while mixing until the connection of the powder with water. Mix rapidly to obtain a homogenous paste (approx. 1 minute).

**Mixing in a vacuum mixer:** Using the proportions shown in the table below progressively pour gypsum into water while mixing to a connection of powder and water (approx. 15 seconds).

After obtaining full vacuum mix the mass for 30 seconds.

### Caution!

**Material for professional use only in technical and dental laboratories as well as dental clinics.**

All written and/or verbal instructions and information given during the presentation by the manufacturer are in accordance with the current level of dental techniques and technology. They should be considered applicable information. No information acquired from any third parties (given their different level of authorisation) releases the user from checking if the product is suitable for the intended particular use. Since the manufacturer has no control over the use of the product, the manufacturer's liability only covers the value of the supplied material.

Any comments relating to our products should be sent to our address.

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